



Laboratory Procedure Manual

Analyte: **Total Folate**

Matrix: **Serum and Whole Blood**

Method: **Microbiologic Assay**

Method No: 4000.10

Revised: June 2020

as performed by: Nutritional Biomarkers Branch (NBB)
Division of Laboratory Sciences (DLS)
National Center for Environmental Health (NCEH)

contact: James L. Pirkle, M.D., Ph.D.
Director, Division of Laboratory Sciences

Important Information for Users

CDC periodically refines these laboratory methods. It is the responsibility of the user to contact the person listed on the title page of each write-up before using the analytical method to find out whether any changes have been made and what revisions, if any, have been incorporated.

Public Release Data Set Information

This document details the Lab Protocol for testing the items listed in the table below for FOLATE_K:

Data file name	Variable name	SAS Label
FOLATE_K	LBDRFO	RBC folate (ng/mL)
	LBDRFOSI	RBC folate (nmol/L)

1. Summary of Clinical Relevance and Principle

A. Clinical Relevance

Folate belongs to the group of water-soluble B vitamins that occur naturally in food. It is required in cellular one carbon metabolism and hematopoiesis. Prolonged folate deficiency leads to megaloblastic anemia. Low folate status has been shown to increase the risk of women of childbearing age to have an offspring with neural tube defects. Low folate status also increases plasma homocysteine levels, a potential risk factor for cardiovascular disease, in the general population. Potential roles of folate and other B vitamins in modulating the risk for diseases (e.g., heart disease, cancer, and cognitive impairment) are currently being studied.

The measurement of total folate (TFOL) provides information on the folate status of the individual. Serum folate is an indicator of short-term status, while red blood cell (RBC) folate is an indicator of long-term status.

B. Test Principle

Microbiological assays (MA) have been used for many years to estimate the concentration of folate in blood and other tissues. In the 1990s O'Broin et al. [1] and Molloy et al. [2] have introduced robust and reliable procedures that use microtitre plates for higher throughput and a cryopreserved antibiotic resistant microorganism to avoid having to work under aseptic conditions. The herein described procedure is an adaptation of the O'Broin et al. method [1] and is used to quantitatively measure serum and RBC folate in human specimens. The method is relatively easy to perform, reliable, and considerably less costly than chromatographic or commercial kit assays.

Diluted serum or whole blood hemolysate is added to an assay medium containing *Lactobacillus rhamnosus* (formerly known as *L. casei*) (NCIB 10463) and all of the nutrients necessary for the growth of *L. rhamnosus* except for folate. The inoculated medium is incubated for ~40-46 hours at 37 °C. Since the growth of *L. rhamnosus* is proportional to the amount of total folate present in serum or whole blood samples, the total folate level can be assessed by measuring the turbidity of the inoculated medium at 590 nm in a microplate reader. The assay is calibrated with 5-methyltetrahydrofolic acid (5MeTHF). If seven plates are processed in a run, 132 patient specimens can be analyzed.

2. Safety Precautions

Consider all whole blood or serum specimens potentially positive for infectious agents including HIV and the hepatitis B virus. We recommend the hepatitis B vaccination series for all analysts working with whole blood and/or serum. Observe universal precautions; wear protective gloves, laboratory coats, and safety glasses during all steps of this method. Discard any residual material (blood samples and assay medium containing bacterial) by autoclaving after analysis is completed. Place disposable plastic, glass, and paper (pipette tips, vials, tubes, gloves, microplates, etc.) that contact serum or blood in a biohazard autoclave bag and keep these bags in appropriate containers until sealed and autoclaved. Use disposable bench diapers during sample preparation and serum/ blood handling and discard them in autoclave pan after use. Also wipe down all contaminated work surface with 10% bleach solution when work is finished. Add bleach into the leftover of folate assay medium containing *L. casei* at final concentration ~10% to kill all the organisms.

Handle acids and bases with extreme care; they are caustic and toxic. Handle organic solvents only in a well-ventilated area or, as required, under a chemical fume hood.

Reagents used in this study include those listed in Section 6. Handle acids and bases with extreme care; they are caustic and toxic. Safety data sheets (SDSs) for these chemicals are readily accessible as hard copies in

the lab. If needed, SDSs for other chemicals can be viewed at <http://www.ilpi.com/msds/index.html> or at <http://intranet.cdc.gov/ossam/workplace-safety/safety-practices/chemical-safety/index.html>

PerkinElmer and BioTek Instrument Inc. provide safety information that should be read before operating the instruments. This information can be found in the Operator's Manual.

3. Computerization; Data System Management

- (A) During sample preparation and analysis, samples are identified by their sample ID. The sample ID is a number that is unique to each sample that links the laboratory information to demographic data recorded by those who collected the sample.
- (B) The raw data file from the microplate reader is collected using the instrument software and stored on the CDC network. The results file (including analyte, sample IDs, replicate count, replicate variation, sample dilution factor, instrument results, final results, data file name, acquisition time, analyst, notes, etc.) is imported into a LIMS database for review of the patient data, statistical evaluation of the QC data, and approval of the results. See **"4000_SOP Computerization and Data System Management"** for a step-by-step description of data transfer, review, and approval.
- (C) For NHANES, data is transmitted electronically on a regular basis (approximately weekly for 3-week turnaround analytes). Abnormal values are confirmed by the analyst, and codes for missing data are entered by the analyst and are transmitted as part of the data file. NCHS makes arrangements for the abnormal report notifications to the NCHS Survey Physician.
- (D) The experiment files (raw data from plate reader) and the result files (Power Export files) are stored on the DLS network and are automatically backed up nightly by ITSO support staff.

4. Specimen Collection, Storage, and Handling Procedures; Criteria for Specimen Rejection

- (A) We recommend that specimen donors fast prior to specimen collection, but fasting is not required.
- (B) Serum folate assays are performed on fresh or frozen serum. RBC folate samples are prepared by diluting 1 part of fresh EDTA whole blood (100 μ L) with 10 parts of 1 g/dL (1%) ascorbic acid solution (1 mL), corresponding to a 1/11 dilution, and freezing the hemolysate promptly, which keeps the folate in the reduced state.
- (C) A minimum of 300 μ L of serum or whole blood hemolysate is needed to do a proper dilution when using automated pipetting. Serum is typically diluted 1/100. Whole blood hemolysate is typically diluted at 1/140 which constitutes a 1/1540 total dilution of whole blood. The dilution factor depends on the population from which the samples are collected. The above dilution factors are appropriate for a population consuming folic acid fortified foods.
- (D) Serum specimens may be collected with regular red-top Vacutainers. Whole blood is collected with lavender-top Vacutainers containing EDTA as an anticoagulant. A hematocrit measurement used for the RBC folate calculations is made at the time of collection. The appropriate amount of serum or whole blood hemolysate is dispensed into a Nalgene cryovial or other plastic screw-capped vial labeled with the participant's ID.
- (E) Specimens collected in the field should be frozen and then shipped on dry ice by overnight mail. Once received, they should be stored at ≤ -20 °C until analyzed. Serum folate and RBC folate are stable for a few weeks if the specimen is frozen at -20 °C before analysis. For long-term storage, specimens should be frozen at -70 °C. Ascorbic acid is typically not added to the serum specimen. Less than three freeze-thaw cycles show only minimal degradation in serum samples and whole blood samples, but freeze-thaw cycles should be avoided where possible due to the sensitive nature of folates.

- (F) Specimens should generally arrive frozen. Specimens received at ambient temperature are rejected. Refrigerated samples may be used when they are brought promptly from the site where the blood was collected. Some methods call for a 90-min incubation to hemolyze the red cells and allow the endogenous folate conjugates to hydrolyze the conjugated pteroylpolyglutamates to pteroylmonoglutamates prior to the assay for RBC folate. However, we have found that if the blood is diluted 1/11 with 1 g/dL ascorbic acid to keep the folate in the reduced state and the hemolysate is frozen promptly, a single freeze-thaw cycle before analysis has the same effect as incubation [3]. Whole blood samples refrigerated for 48 hours before hemolysis give same results as samples lysed immediately after collection.
- (G) Diurnal variation is not a major consideration. Hemolyzed serum specimens should be interpreted with caution because they may have falsely elevated values. Folate specimens exposed to light for longer than 8 hours may undergo 10-20% degradation [4]. Therefore, specimens intended for folate analysis should be processed under yellow light and stored frozen promptly if analysis is not to be performed within 8 hours of collection.
- (H) Specimen handling conditions are outlined in the DLS Policies and Procedures Manual. The protocol discusses collection and transport of specimens and the special equipment required. Samples thawed and refrozen less than three times are not compromised. If there is more than one test of interest in the specimen and it needs to be divided, the appropriate amount of blood or serum should be transferred into a Nalgene cryovial labeled with the Participant's ID; avoid cross-contamination.

5. Procedures for Microscopic Examinations; Criteria for Rejection of Inadequately Prepared Slides

Not applicable for this procedure.

6. Preparation of Reagents, Calibration (Standards), Controls, and All Other Materials; Equipment and Instrumentation

A. Standards and Chemicals

With some exceptions, a material listed herein may be substituted with an equivalent product from a different manufacturer provided that it meets or exceeds the specifications of the product listed.

- 5MeTHF (Merck Cie, AG, IM Laternenacker 5, Schaffhausen 8200 [CH] Switzerland)
- Ascorbic acid (Sigma, St. Louis, MO, USA)
- Sodium ascorbate (Sigma)
- Himedia Folic acid casei medium (M543)
- Chloramphenicol (Sigma)
- Tween-80 (Sigma)
- Manganese sulfate (Sigma)
- *Lactobacillus Rhamnosus (formerly called Casei)* (ATCC 27773 or called NCIB 10463, American Type Culture Collection, Manassas, VA, USA)
- Sodium azid (Sigma)
- Glycerol (Sigma)

B. Reagent Preparation

Though each reagent preparation specifies a total volume of reagent prepared, these directions may be scaled up or down to prepare larger or smaller quantities if desired.

1) Sodium ascorbate (0.5%)

Sodium ascorbate solution (0.5 g/dL) for diluting samples and standard.

Two and half grams of sodium ascorbate is dissolved in 500 mL of deionized water. This solution should be freshly prepared before each assay.

2) Folic Acid Casei medium inoculated with *L. casei* (folate free)

L. casei assay medium inoculated with microorganism (folate free).

To prepare 200 mL of assay medium: add 14.1 g of Folic Acid Casei Medium, 6 mg of chloramphenicol (or 2 mL of a 3 mg/mL chloramphenicol stock solution) and 60 μ L of Tween-80 to 200 mL of ultrapure water, heat to boil for 2-3 min with stirring. Cool down to \sim 37 $^{\circ}$ C, then add 30 mg of manganese sulfate (or 1 mL of a 30 mg/mL manganese sulfate stock solution) and 150 mg of ascorbic acid (or 1 mL of a 150 mg/mL ascorbic acid stock solution), and keep stirring until all the chemicals are dissolved. Thaw one vial of frozen *L. casei* as quickly as possible (use running water) and add 600-700 μ L into 200 mL of assay medium, keep stirring slowly. About 25 mL of assay medium is needed per microplate.

3) Ascorbic acid stock solution (150 mg/mL)

Dissolve 15 g of ascorbic acid in 100 mL of deionized water. Aliquot 1 mL/vial and store at -70 $^{\circ}$ C. Add 1 mL into 200 mL of Folic Acid Casei medium. Make fresh stock solution every 2 years. (Appendix D_F)

4) Manganese sulfate stock solution (30 mg/mL)

Dissolve 3.6 g of manganese sulfate in 120 mL of deionized water, stir thoroughly (\sim 20 min). Aliquot 1 mL/vial and store at -70 $^{\circ}$ C. Add 1 mL into 200 mL of Folic Acid Casei medium. Make fresh stock solution every 2 years.

5) Chloramphenicol stock solution (3 mg/mL)

Dissolve 600 mg of chloramphenicol in 4 mL of ethanol and then make up to 200 mL with deionized water, aliquot 2 mL into cryovials and keep at -70 $^{\circ}$ C freezer. Add 2 mL of stock solution into 200 mL of Folic Acid Casei medium. Make fresh stock solution every 2 years.

6) *L. casei* growth medium (with folate)

L. casei growth medium (with folate to create new inoculum).

To prepare 200 mL of growth medium: Add 9.4 g of Folic Acid Casei Medium, 40 mg of chloramphenicol, 40 μ L of Tween-80 to 200 mL of deionized water. Heat to boil for 2-3 min. Cool down to \sim 37 $^{\circ}$ C, add 100 mg of ascorbic acid, 300 μ L of folic acid stock solution (100 ng/mL), stir to completely dissolve, sterilize the medium by either autoclaving or filtering through a 0.2 μ m filter. Aliquot 20 mL into sterile 50-mL tubes and keep at -20 $^{\circ}$ C.

7) Ascorbic acid (1%)

Ascorbic acid (1 g/dL) for hemolysis of whole blood samples.

Dissolve 1 g of ascorbic acid in 10 mL of deionized water. Prepare fresh solution before each use.

8) Blocking solution

Blocking solution for color control. Sodium azide is used as a blocking solution. Five μL of a 3 g/dL solution are added to each control well. Alternatively, the disinfectant "Stericol" can be used at a 1/30 dilution with ultrapure water (10 μL /control well).

C. Standards Preparation

The concentrations of the folate stock solution is calculated using molar absorptivity. Information on the absorption maximum, absorption coefficient, and formula to calculate the concentration for the stock solution is provided in **Appendix B**. Use Class A volumetric glassware where a volumetric flask is specified.

1) 5-MethylTHF primary stock solution (< 200 $\mu\text{g}/\text{mL}$)

Prepare a primary stock solution in a volumetric flask by dissolving an accurately known mass (± 0.1 mg) of the pure solid compound in degassed and filtered 20 mM phosphate buffer (pH 7.2, containing 0.1% cysteine), targeting a final concentration of ~ 100 $\mu\text{g}/\text{mL}$ (e.g., 5 mg in 50 mL). Vortex briefly to help dissolve the contents and make up to final volume.

Remove a small aliquot (1 mL) of the primary stock solution to a microcentrifuge vial to determine its concentration by UV spectrophotometry. To the remaining stock solution add ascorbic acid powder to a final concentration of 1%. From the aliquot you removed, prepare two dilutions (e.g., 1/10 and 1/20), each in duplicate, measure the UV absorbance at the peak maximum using scan analysis against phosphate buffer as a blank, and calculate the primary stock solution concentration (**Appendix B**). For 5-methylTHF, the ratio of absorbance at 290/245 nm can be monitored (simple reads analysis at each wavelength) to ensure that no oxidation takes place. This ratio should exceed 3.3.

Aliquot the remainder of the primary stock solution into cryovials that are stored at -70 $^{\circ}\text{C}$. The primary stock solution is stable for at least 2 years.

2) 5-MethylTHF microbiological assay stock solution (1 $\mu\text{mol}/\text{L}$)

Based on the exact concentration determined on UV spectrophotometry, dilute 5-MethylTHF primary stock solution with 0.5% degassed ascorbic acid in a 100-mL volumetric flask to get final concentration 1 $\mu\text{mol}/\text{L}$. Aliquot 500 μL into labeled cryovial and freeze the aliquots at -70 $^{\circ}\text{C}$. Prepare a fresh MA stock solution from primary stock solution every 1.5 year (Appendix D_G).

D. Preparation of Quality Control Materials

Three levels of serum or whole blood hemolysate (whole blood diluted 1/11 with 1% ascorbic acid solution) quality control pools are used as bench QCs in every run to detect any problems with the microbiologic assay. Since we use the same pools for the folate microbiologic assay and the folate LC/MS/MS assay, we prescreen potential material with the LC/MS/MS assay to determine the concentrations of the individual folate forms. We then either spike in individual folate primary standards or dilute pools with a protein diluent or PBS solution to achieve desired concentrations. This is described in detail in the folate LC/MS/MS procedure.

Pool specimens are aliquoted into 2.0 mL Nalgene cryovials, capped and frozen. The QC pools are stored at -70 $^{\circ}\text{C}$ and are stable for at least 2 years. Means plus range limits for all pools are established by analyzing the QC pools in at least 20 runs.

E. Preparation of Cryoprotected Organism

- (1) Add one vial of freeze-dried *L. casei* into 20 mL of organism growth medium and incubate at 37°C for 24 hours.
- (2) Transfer 100–300 µL of the 24-hour culture into another 20 mL of organism growth medium and incubate at 37°C for 24 hours.
- (3) Repeat step 2 for the third 24-hour incubation.
- (4) Inoculate different amount of active culture in duplicates for optimization of response. Add 500 µL, 1 mL and 2 mL of Step 3 culture into 20 mL of fresh growth medium and incubate at 37 °C. Measure OD at 590 nm from one of the duplicate cultures at different incubation time. Record the log growth phase (~18-20 hours). Mix the log phase culture by 50/50 with 80% glycerol (sterilized by autoclaving). Aliquot the mixture in sterile cryovials (1 mL/vial) and store at -70 °C.
- (5) Run a standard plate using the new inoculum and compare to a standard plate using the previous inoculum. The OD for the blank should be ~0.1, for the highest calibrator ~1, and the curve should have the expected polynomial 3rd degree shape.

F. Equipment

In the case of simple laboratory instrumentation, a product listed herein may be substituted with equivalent product from a different manufacturer provided that it meets or exceeds the specifications of the product listed. Equivalent performance must also be demonstrated in accordance with DLS policies and procedures when multiple analysis systems are used in parallel, even if they are of the exact same type.

- (1) PowerWave microplate reader and GEN5 software (Bio-Tek Instruments, Inc, Winooski, VT, USA)
- (2) JANUS™ Automated Workstation (PerkinElmer, MA, USA)
- (3) Precision Incubator 6842: 37 °C (Fisher, Pittsburgh, PA)
- (4) Thermo Microplate Heat Sealer (Thermo Scientific, ALPS 50 V)
- (5) Rotator-Genie (Scientific Industries #7400-2100)
- (6) Digiflex CX (Titertek #0603310, Huntsville, AL)
- (7) Stirring Hotplate (Fisher # 11-100-16SH, 11-100-49SH, Suwanee, GA and Thermolyne SP46925)
- (8) Balance (Mettler Toledo, AG-104)
- (9) Vortex – Genie 2 (VWR, Suwanee, GA)
- (10) Eppendorf adjustable pipettes (1-200 µL and 100-1000 µL); Eppendorf Repeater Plus pipette; 8-channel pipetter (Rainin)

G. Other Materials

With some exceptions, a material listed herein may be substituted with equivalent product from a different manufacturer provided that it meets or exceeds the specifications of the product listed. Equivalent performance must be demonstrated experimentally in accordance with DLS policies and procedures.

- (1) Ultrapure water (18.2 M-Ohm from an AquaSolutions water purification system)

- (2) Pipette tips: 1-200 μ L (Eppendorf, fits up to 200 μ L pipettes) and 200 -1000 μ L (Eppendorf, fits between 100 μ L and 1000 μ L pipettes), Combitip plus 5 mL and Combitip plus 0.5 mL for Eppendorf Repeater Plus (Brinkmann Instruments, Inc., Westbury, NY), 250 μ L 8-channel pipette tips (#GP-L250, Rainin Instrument Co., Inc., Woburn, MA)
- (3) Dehydrated alcohol, USP (Midwest Grain Products of Illinois, Pekin, IL)
- (4) Vinyl examination gloves (Travenol Laboratories, Inc., Deerfield, IL)
- (5) Biohazard autoclave bags (Curtin-Matheson Scientific, Inc., Atlanta, GA)
- (6) Bleach (10% sodium hypochlorite solution) – any vendor
- (7) 12 x 75 mm Disposable glass culture tubes (Corning Glassworks, NY)
- (8) Kay-Dry paper towels and Kim-Wipe tissues (Kimberly-Clark Corp., Roswell, GA)
- (9) Volumetric flask (50 mL, any vendor)
- (10) Beakers (50 mL, 400 mL, 600 mL, any vendor)
- (11) 96-well microplate (Nunc, 12-565-361, Waltham, MA)
- (12) Thermal Seal 2 Films for PCR (Thomas Scientific, TS2-100)
- (13) Pipetter solution basin (VWR, # 21007-970, Suwanee, GA)

7. Calibration and Calibration Verification Procedures

A. Method Calibration

This assay uses an aqueous calibration curve with 5MeTHF (Merck Cie [Eprova]) as calibrator. Calibrators are carried through the entire sample processing procedure. Calculation of folate concentrations in QCs and unknown patient samples is based on results obtained from a daily 11-point calibration curve (0, 0.05, 0.10, 0.15, 0.20, 0.30, 0.40, 0.50, 0.60, 0.80, 1.0 nmol/L) using a polynomial regression (third degree) since the growth response of the microorganism is not linear.

Calibration verification is conducted at least twice a year using international reference materials. For details, see **4000_SOP Calibration and Calibration Verification FOL MA**.

The National Institute of Standards and Technology (NIST) has released in 2005 a new standard reference material for homocysteine and folate in human serum, SRM 1955. This material was characterized by various methods used at NIST and the CDC. Good agreement for total folate was found for the three levels of SRM 1955 between the CDC LC-MS/MS and microbiologic assay (level 1: 6.0 vs. 5.4; level 2: 13 vs. 14; level 3: 41 vs. 42). We routinely use this SRM material at least twice a year during calibration verification and any time we experience assay problems.

In 2006, the National Institute for Biological Standards and Control (NIBSC) issued the first WHO certified reference material (lyophilized) for folate in human serum (03/178, established 2004). The folate concentration in this material has been certified by LC-MS/MS. The NIBSC also provides a WHO certified reference material (lyophilized) for whole blood folate (95/528, established 1996). The folate concentration in this material has been determined by consensus value assignment, but is mainly representative for the microbiologic assay. Both materials are used at least twice a year during calibration verification and any time we experience assay problems.

We participate in the UK NEQAS Haematinics survey external proficiency testing program twice a year. Details can be found in the proficiency testing form. For general information on the handling, analysis, review, and reporting of proficiency testing materials see “**NBB_SOP Proficiency Testing Procedure**”.

Information on how the microorganism responds to various folate calibrators and how the microbiologic assay compares to other assays is presented in **Appendix C**.

Results from a series of in-house ruggedness testing experiments designed to assess how much method accuracy changes when certain experimental parameters are varied are presented in **Appendix D**.

B. Instrument Calibration

1) PerkinElmer Janus Liquid Handler

Once a year a PerkinElmer service engineer performs preventative maintenance. The in-house calibration verification of the Janus Liquid handlers is scheduled every 6 months either using yellow dye method or gravimetric method. If necessary, extra calibration verification is performed as part of troubleshooting

2) BioTek Microplate Reader

Microplate readers are tested twice a year by measuring CAP proficiency testing samples and performing a reader system test.

3) Pipettes and Digiflex

Calibration is performed annually by a certified company. Every six months, calibration verification is performed by the analyst gravimetrically, using a calibrated analytical balance.

4) Balances

Calibration is performed annually by a certified company. Calibration verification is performed by the analyst as needed using certified weights.

8. Procedure Operating Instructions; Calculations; Interpretation of Results

A. Preliminaries

- (1) Thaw frozen serum or whole blood hemolysate samples, serum or whole blood hemolysate quality control samples, and a vial of 5MeTHF MA stock solution (1 $\mu\text{mol/L}$) and let them reach room temperature
- (2) Place QC samples, unknown samples and blank vials in 4x12 Janus Liquid Handler Racks following MA Gen5 plate layout
- (3) Label the same number of 75x12-mm glass tubes by 1, 2, 3..., and place them in 4x12 Janus Liquid Handler Racks
- (4) Thoroughly vortex QCs, unknown specimens and standard before pipetting
- (5) Prepare assay medium inoculated with microorganism and 0.5% sodium ascorbate

B. Dilution of Samples and Calibrator

- (1) To dilute serum QCs and unknown samples at 1:100, add 1485 μL of 0.5 g/dL sodium ascorbate into each 75x12-mm glass tube using a Digiflex liquid dispenser, and then add 15 μL of serum QC or

unknown samples into each tube using the JANUS Automated Workstation. When completed, cover the tubes with plastic wrap and thoroughly vortex the rack with all tubes on a multi-tube shaker to mix.

- (2) To dilute hemolysed whole blood QC and unknown samples (whole blood diluted 1:11 with 1% ascorbic acid) at 1:140, add 2085 μL of 0.5 g/dL sodium ascorbate into each 75x12-mm glass tube using a Digiflex liquid dispenser, and then add 15 μL of whole blood hemolysate into each tube using the JANUS Automated Workstation. When completed, cover the tubes with plastic wrap and thoroughly vortex the rack with all tubes on a multi-tube shaker to mix.
- (3) Calibrator dilution (to obtain working solution I and II)
 - a) Diluted 5MeTHF MA stock solution: Dilute 5MeTHF stock solution (1 $\mu\text{mol/L}$) at
 - b) 1:5 (200 nmol/L) by adding 100 μL of stock solution (1 $\mu\text{mol/L}$) to 400 μL of 0.5 g/dL sodium ascorbate.
 - c) Working solution I: dilute 50 μL of diluted 5MeTHF stock III with 0.5 g/dL sodium ascorbate in a 50 mL volumetric flask and make up to volume (1:1000 dilution; 200 pmol/L).
 - d) Working solution II: dilute 250 μL of diluted 5MeTHF stock III with 0.5 g/dL sodium ascorbate in a 50 mL volumetric flask and make up to volume (1:200 dilution; 1 nmol/L).

C. Addition of Standard, QCs, and Unknown Samples into 96-well Microplates

We conduct this assay by using 4 replicates for each QC and unknown sample at 2 different dilutions (2 replicates/dilution) and 8 replicates for each standard concentration. Fully automatic pipetting is performed by the JANUS Automated Workstation operated by the WinPrep software. The total pipetting time is ~2 hours per run (for 7 plates). The total time for each assay is ~5 hours. A total of 22 unknown patient samples can be analyzed per sample plate (if 7 plates are run in an assay).

1) Plate 1 – 5MeTHF calibration plate

- a) Add 200 μL of folic acid casei medium inoculated with *L. casei* into each well of 96-well microplate (see Calibration Plate - Table 1).
- b) Add a different amount of 0.5 g/dL sodium ascorbate into each column of the 96-well plate (see Calibration Plate -Table 2)
- c) Add a different amount of 5MeTHF standard working solution I and II into each well (see Calibration Plate - Table 3). The calibration curve is made up of 11 different concentration points. The total standard volume is 100 μL in each well.
- d) Optional step: Add 5 μL of 3 g/dL sodium azide to the first column for the M590 correction (used to generate the calibration curve for whole blood, but ignored when the calibration curve for serum is generated). This step is currently not used for NHANES whole blood samples because of high folate concentrations and resulting higher dilution.
- e) Seal the plate very tightly using Thermal Sealing membrane and Heat Plate Sealer (150 °C for 5 seconds). Do not invert the plates.
- f) Incubate the plate with other sample plates together at 37 °C for 40-46 hours depending on the lot of the medium.

Calibration Plate -Table 1: Add 200 μL *L. casei* assay medium inoculated with microorganism

#1	1	2	3	4	5	6	7	8	9	10	11	12
A	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
B	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
C	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
D	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
E	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
F	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
G	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
H	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL

Calibration Plate - Table 2: Add 0.5 g/dL sodium ascorbate

#1	1	2	3	4	5	6	7	8	9	10	11	12
A	100ul	100ul	75ul	50ul	25ul	0ul	70ul	60ul	50ul	40ul	20ul	0ul
B	100ul	100ul	75ul	50ul	25ul	0ul	70ul	60ul	50ul	40ul	20ul	0ul
C	100ul	100ul	75ul	50ul	25ul	0ul	70ul	60ul	50ul	40ul	20ul	0ul
D	100ul	100ul	75ul	50ul	25ul	0ul	70ul	60ul	50ul	40ul	20ul	0ul
E	100ul	100ul	75ul	50ul	25ul	0ul	70ul	60ul	50ul	40ul	20ul	0ul
F	100ul	100ul	75ul	50ul	25ul	0ul	70ul	60ul	50ul	40ul	20ul	0ul
G	100ul	100ul	75ul	50ul	25ul	0ul	70ul	60ul	50ul	40ul	20ul	0ul
H	100ul	100ul	75ul	50ul	25ul	0ul	70ul	60ul	50ul	40ul	20ul	0ul

Calibration Plate - Table 3: Add 5-methylTHF working solution I and II

#1	working solution I						working solution II					
	1	2	3	4	5	6	7	8	9	10	11	12
A	0ul	0ul	25ul	50ul	75ul	100ul	30ul	40ul	50ul	60ul	80ul	100ul
B	0ul	0ul	25ul	50ul	75ul	100ul	30ul	40ul	50ul	60ul	80ul	100ul
C	0ul	0ul	25ul	50ul	75ul	100ul	30ul	40ul	50ul	60ul	80ul	100ul
D	0ul	0ul	25ul	50ul	75ul	100ul	30ul	40ul	50ul	60ul	80ul	100ul
E	0ul	0ul	25ul	50ul	75ul	100ul	30ul	40ul	50ul	60ul	80ul	100ul
F	0ul	0ul	25ul	50ul	75ul	100ul	30ul	40ul	50ul	60ul	80ul	100ul
G	0ul	0ul	25ul	50ul	75ul	100ul	30ul	40ul	50ul	60ul	80ul	100ul
H	0ul	0ul	25ul	50ul	75ul	100ul	30ul	40ul	50ul	60ul	80ul	100ul

2) Plate #2 to #7: Serum QCs and Unknown Samples

- a) Add 200 µL of folic acid casei medium inoculated with *L. casei* into each well (see Sample Plate - Table 4)
- b) Add 50 µL of 0.5 g/dL sodium ascorbate into the wells that will have 50 µL diluted sample; don't add sodium ascorbate into the wells that will have 100 µL diluted sample (see Serum Plate - Table 5).
- c) Add 50 and 100 µL of the 1:100 diluted serum QCs and unknown samples to the corresponding wells (see Sample Plate -Table 6). Total sample volume is 100 µL in each well. A total of 24 samples can be analyzed per plate including QCs and 1 blank. The last sample in each plate is always a blank.
- d) Seal the plates very tightly using Thermal Sealing membrane and Heat Plate Sealer (150 °C for 5 seconds). Do not invert the plates.
- e) The layouts of Plate #3-#7 are the same as for plate #2; each plate must contain at least one level of QC and a blank. Incubate the plates at 37 °C for ~40-46 hours, depending on the lot of the medium.

Serum Plate – Table 4: Add 200 µL *L. casei* assay medium inoculated with microorganism

#2	1	2	3	4	5	6	7	8	9	10	11	12
----	---	---	---	---	---	---	---	---	---	----	----	----

	QC low 1		Sample 2		Sample 6		Sample 10		Sample 14		Sample 18	
A	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
B	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
	QC med 1		Sample 3		Sample 7		Sample 11		Sample 15		Sample 19	
C	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
D	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
	QC high 1		Sample 4		Sample 8		Sample 12		Sample 16		Sample 20	
E	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
F	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
	Sample 1		Sample 5		Sample 9		Sample 13		Sample 17		Sample 21-blank	
G	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
H	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL

Sample Plate – Table 5: Add 0.5 g/dL sodium ascorbate

#2	1	2	3	4	5	6	7	8	9	10	11	12
	QC low 1		Sample 2		Sample 6		Sample 10		Sample 14		Sample 18	
A	50uL		50uL		50uL		50uL		50uL		50uL	
B	50uL		50uL		50uL		50uL		50uL		50uL	
	QC med 1		Sample 3		Sample 7		Sample 11		Sample 15		Sample 19	
C	50uL		50uL		50uL		50uL		50uL		50uL	
D	50uL		50uL		50uL		50uL		50uL		50uL	
	QC high 1		Sample 4		Sample 8		Sample 12		Sample 16		Sample 20	
E	50uL		50uL		50uL		50uL		50uL		50uL	
F	50uL		50uL		50uL		50uL		50uL		50uL	
	Sample 1		Sample 5		Sample 9		Sample 13		Sample 17		Sample 21-blank	
G	50uL		50uL		50uL		50uL		50uL		50uL	
H	50uL		50uL		50uL		50uL		50uL		50uL	

Sample Plate – Table 6: Add diluted serum QC and unknown samples

#2	1	2	3	4	5	6	7	8	9	10	11	12
	QC low 1		Sample 2		Sample 6		Sample 10		Sample 14		Sample 18	
A	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL
B	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL
	QC med 1		Sample 3		Sample 7		Sample 11		Sample 15		Sample 19	
C	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL
D	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL
	QC high 1		Sample 4		Sample 8		Sample 12		Sample 16		Sample 20	
E	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL
F	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL
	Sample 1		Sample 5		Sample 9		Sample 13		Sample 17		Sample 21-blank	
G	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL
H	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL	50uL	100uL

3) Plate #2 to #7: WB QCs and WB Unknown Samples

- a) For samples that have relatively high folate concentration, we use higher dilution factor of 1:140 for whole blood hemolysate samples. We have found that the hemoglobin color has no effect on OD 590 after incubation, even at 1:70 dilution, therefore we decided not to use color control and the whole blood samples have exactly the same plate layout as serum samples.
- b) For other studies where the whole blood samples have lower folate concentrations, a lower dilution has to be used (i.e., 1:40) and the color control with sodium azide is needed. The plate layouts are shown below.

- c) Add 200 µL of folic acid casei medium inoculated with *L. casei* into each well (see Whole Blood Plate – Table 7).
- d) Add 50 µL of 0.5 g/dL sodium ascorbate into the wells that will have 50 µL diluted sample; don't add sodium ascorbate into the wells that will have 100 µL diluted sample (see Whole Blood Plate – Table 8).
- e) Add 50 µL and 100 µL of the 1:140 diluted hemolysate QCs and unknown samples to the corresponding wells (see Whole Blood plate –Table 9).
- f) Add 5 µL of 3 g/dL sodium azide to the third column of each sample (column 3, 6, 9, 12) to generate a color control (there won't be any growth in these wells).
- g) Seal the plates very tightly using Thermal Sealing membrane and Heat Plate Sealer (150 °C for 5 seconds). Do not invert the plates.
- h) The layouts of Plate #3-#7 are the same as for plate #2; each plate must contain at least one level of QC and a blank. Incubate the plates at 37 °C for ~40-46 hours, depending on the lot of the medium.

WB sample plate – Table 7: Add 200 µL *L. casei* assay medium inoculated with microorganism

	1	2	3	4	5	6	7	8	9	10	11	12
	WB QC low 1			Sample 2			Sample 6			Sample 10		
A	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
B	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
	WB QC med 1			Sample 3			Sample 7			Sample 11		
C	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
D	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
	WB QC high 1			Sample 4			Sample 8			Sample 12		
E	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
F	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
	Sample 1			Sample 5			Sample 9			Sample 13-blank		
G	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL
H	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL	200uL

WB sample plate – Table 8: Add 0.5% sodium ascorbate

	1	2	3	4	5	6	7	8	9	10	11	12
	WB QC low 1			Sample 2			Sample 6			Sample 10		
A	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL
B												
	WB QC med 1			Sample 3			Sample 7			Sample 11		
C	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL
D												
	WB QC high 1			Sample 4			Sample 8			Sample 12		
E	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL
F												
	Sample 1			Sample 5			Sample 9			Sample 13-blank		
G	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL
H												

WB sample plate – Table 9: Add diluted WB hemolysates

	1	2	3	4	5	6	7	8	9	10	11	12
	WB QC low 1			Sample 2			Sample 6			Sample 10		
A	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL
B	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL
	WB QC med 1			Sample 3			Sample 7			Sample 11		
C	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL
D	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL
	WB QC high 1			Sample 4			Sample 8			Sample 12		
E	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL
F	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL
	Sample 1			Sample 5			Sample 9			Sample 13-blank		
G	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL	50uL
H	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL	100uL

D. Instrument & Software Setup for PowerWave Microplate Reader

The Bio-Tek PowerWave microplate reader uses the GEN5 software. Templates of various folate MA protocols (for different number of plates) have been saved and can be used to efficiently create an experiment file (saved by date) by entering the sample IDs and dilution factors. This should be done as soon as the assay setup is finished and the plates are put into the incubator.

Table 10: Parameters in GEMS

GEN5 Parameter	Setting
Reading Type	End Point
Wave Length	590 nm
Shaking Intensity	0
Shaking Duration	0
Temperature control	No
Data Interpolate	M590 or M590 Corr (if needed)
X Axis	Lin
Y Axis	Lin
Curve Fit	Polynomial Regression, Degree 3

For different plates, the layouts vary based on different samples analyzed. The calibration curve has the following concentrations: 0, 0.05, 0.1, 0.15, 0.2, 0.3, 0.4, 0.5, 0.6, 0.8, 1.0 nmol/L. Specific dilution factors have to be used for QCs and unknown samples (see Table 11).

Table 11: Dilution factors for serum and whole blood hemolysate samples

Sample		Dilution factor (100 uL added into plate)	Sodium Ascorbate (uL)	Serum or whole blood hemolysate (uL)	Final volume (uL)	Comment code	Dilution factor in GEN5 software
Serum	Routine Dilution	100	1485	15	1500	none	100 w/ 100 uL 200 w/ 50 uL
	GEN5 raw concentration ~ < 0.05 nmol/L	50	735	15	750	115	50 w/ 100 uL 100 w/ 50 uL
		30	435	15	450	115	25 w/ 100 uL 50 w/ 50 uL
	GEN5 raw concentration > 1.00 nmol/L	200	1485	Predilute at 1/2, then add 15 uL	1500	97	200 w/ 100 uL 400 w/ 50 uL
		400	1485	Predilute at 1/4, then add 15 uL	1500	97	400 w/ 100 uL 800 w/ 50 uL
	Whole blood hemolysate (1/11 diluted WB)	Routine Dilution	140	2085	15	2100	none
GEN5 raw concentration ~ < 0.05 nmol/L		70	1035	15	1050	115	770 w/ 100 uL 1540 w/ 50 uL
		50	735	15	750	115	385 w/ 100 uL 770 w/ 50 uL
GEN5 raw concentration > 1.00 nmol/L		280	2085	Predilute at 1/2, then add 15 uL	2100	97	3080 w/ 100 uL 6160 w/ 50 uL
		560	2085	Predilute at 1/4, then add 15 uL	2100	97	6160 w/ 100 uL 12320 w/ 50 uL

E. Collection of data and calculations of results

Turn on PowerWave microplate reader before use and let it warm up for ~30 min.

Remove all plates from the incubator after ~40-46 hours, and let them cool to room temperature while mixing them thoroughly by rotating them on a 360 degree rotator. The mixing time is at least 10 minutes. Plates are typically read directly after mixing. However, if that is not possible, plates can be placed at room temperature for ~ 1-2 hours or in the refrigerator for up to 2 days and reading can be delayed. Prior to reading, plates have to be brought up to room temperature and mixed thoroughly.

1) Collect calibration data and construct calibration curve

Take off the sealing membrane from Plate 1 and let it sit on the bench for ~1-2 min to wait for all air bubbles to dissipate. The same procedure is used for every plate before collecting data. Do not take off the sealing membranes from more than 1 plate at a time. If plates sit for too long on the bench prior to reading the OD, there might be some uneven settlement of the organisms leading to inaccurate results.

In GEN5 software, open the experiment, go to plate 1 screen (calibration plate), and press "Read". The reading takes ~ 1 minute. The raw OD results are automatically collected and also the calibration curve is automatically generated; save the data by clicking "Save".

After reading is complete, analyst visually check the calibration plate and mask wells that have obvious contamination, leaking, dark color, less volume, or where the OD is greatly different from the average of 8 replicates. Analyst checks OD readings at each calibration concentration point. The calibration curve represents the following concentrations: 0, 0.05, 0.1, 0.15, 0.2, 0.3, 0.4, 0.5, 0.6, 0.8 and 1.0 nmol/L.

2) Collect sample data and calculate the results

Put each sample plate into the microplate reader after checking for bubbles. Open the corresponding plate screen, press "Read", and save the raw OD results.

After reading is complete, analyst examines the results in detail as described above (including blank contamination, leaking wells, dark color wells, volumes in wells). A review is also performed for number of replicates, CV of replicates, dilution factors, formulas, results >high cal, results < cutoff value (raw folate concentration <0.05 nmol/L), and unusual results to determine which samples require repeating and which samples require special comment codes (i.e., hemolyzed sample, no ascorbic acid solution added to whole blood, potential antibiotic interference). Appropriate notes are made. See section 10.c. on "Sample QC criteria".

Calculation of RBC Folate Concentration:

Whole blood hemolysate folate results are multiplied by 11, the dilution factor of the whole blood. The serum folate values (multiplied by 1.0 minus the hematocrit [Hct] expressed as a decimal) are subtracted, and the resulting value is divided by the Hct to yield RBC folate in nmol/L RBC. If a Hct value is not available for a patient sample or QC sample, a Hct of 40% can be assumed for calculation. If a serum folate value is not available, a value of 18 nmol/L can be assumed for calculation. We recommend use of the correction for serum folate level and Hct because it provides the most accurate reflection of folate body stores.

$$\text{RBC folate, } \frac{\text{nmol}}{\text{L}} = \frac{(\text{Whole blood hemolysate folate} * 11) - \text{Serum folate} \left(1 - \frac{\text{Hct}}{100}\right)}{\frac{\text{Hct}}{100}}$$

F. Special Procedure Notes - CDC Modifications

This method is based on the method described by O'Broin et al. [1] and Molloy et al. [2].

9. Reportable Range of Results (AMR – Analytical Measurement Range)

The reportable range and the LOD of this assay are dependent on the dilution factor for unknown samples. There is no known maximum acceptable dilution. When possible, avoid small volume pipetting and minimize use of serial dilutions when generating diluted samples. For a population with good folate status, the typical dilution of serum samples is 1:100, resulting in a reportable range of 10.0–100 nmol/L:

- 0.05 nmol/L [lowest calibrator] * 100 [dilution factor] * 2 [to ensure 4 valid results from the 50 and 100 µL addition of diluted sample] = 10.0 nmol/L
- 1 nmol/L [highest calibrator] * 100 [dilution factor] = 100 nmol/L.

Serum samples with a raw concentration less than 0.05 nmol/L or greater than 1 nmol/L are repeated with lower or higher dilution, respectively.

The lowest possible dilution of serum samples is 1:20 resulting in an LOD of 2.0 nmol/L:

- 0.05 nmol/L * 20 * 2 [to ensure 4 valid results from the 50 and 100 uL addition of diluted sample] = 2.0 nmol/L.

A serum folate result of <7 nmol/L (3 ng/mL) is considered to represent potential folate deficiency and needs to be repeated for confirmation.

The typical dilution of whole blood samples is 1:1540 (1:11 dilution of whole blood to hemolysate * 1/140 dilution of the hemolysate), resulting in a reportable range of 154-1540 nmol/L:

- $0.05 \text{ nmol/L [lowest calibrator]} * 1540 \text{ [dilution factor]} * 2 \text{ [to ensure 4 valid results from the 50 and 100 uL addition of diluted sample]} = 154 \text{ nmol/L}$
- $1 \text{ nmol/L [highest calibrator]} * 1540 \text{ [dilution factor]} = 1540 \text{ nmol/L.}$

Whole blood samples with a raw concentration less than 0.05 nmol/L or greater than 1 nmol/L are repeated with lower or higher dilution, respectively.

The lowest possible dilution of whole blood samples is 440 (1:11 dilution of whole blood to hemolysate * 1:40 dilution of the hemolysate) resulting in an LOD of 44 nmol/L. Assuming a hematocrit of 40%, this would correspond to a RBC folate concentration of 110 nmol/L RBC:

- $0.05 \text{ nmol/L} * 440 * 2 \text{ [to ensure 4 valid results from the 50 and 100 uL addition of diluted sample]} = 44 \text{ nmol/L.}$
- $44 \text{ nmol/L} / 40\% = 110 \text{ nmol/L}$

A RBC folate result of <305 nmol/L RBC (140 ng/mL RBC; 127 nmol/L whole blood assuming a hematocrit of 40%) is considered to represent potential folate deficiency and has to be repeated for confirmation.

Dilution linearity

Dilution experiments for serum and whole blood samples showed good dilution linearity when serum samples were diluted between 1/25 and 1/200 ($y=0.9933x+0.2156$, $R^2=0.9987$, $n=3$ subjects) and whole blood hemolysate samples were diluted between 1/70 and 1/280 ($y=1.0159x-4.8306$, $R^2=1$, $n=2$ subjects).

10. Quality Control (QC) Procedures

A. Blind Quality Controls

Blind QC specimens can be inserted into the mix of patient specimens. These QC specimens are generally prepared at two levels that would be encountered in patient samples; the labels used are identical to those used for patient samples. One blind QC specimen randomly selected for concentration is included at a randomly selected location in every 20 specimens analyzed.

B. Bench Quality Controls

Bench QC specimens are processed from three serum and/or whole blood hemolysate pools, which represent low, medium and high folate levels (~25, ~40, and ~60 nmol/L for serum folate and ~300, ~400, and ~600 nmol/L for whole blood folate), approximating the 25th, 50th, and 75th percentile in the U.S. population. The QC specimens are processed in the same manner as patient samples and analyzed in 4 replicates each at the front and back of each run. Each sample plate should contain at least one QC specimen.

The QC results are checked after each run using of a multi-rule quality control program [5]. A QC program written in SAS is available from the DLS Quality Assurance Officer and should be used to apply these rules to QC data and to generate Shewhart QC charts. For this assay, the mean of the 4 replicates represents one QC result. Thus, there are two QC results (front and back QC) that are evaluated.

- (1) If all three QC run means are within $2 S_m$ limits and individual results are within $2 S_i$ limits, accept the run ("in control")
- (2) If one of the three QC run means is outside a $2 S_m$ limit, reject run if ("out of control"):
 - a) 1 3S Rule—Run mean is outside a $3 S_m$ limit or

- b) 2 2S Rule—Two or more of the three run means are outside the same $2 S_m$ limit or
 - c) 10 X-bar Rule—Current and previous nine run means are on the same side of the characterization mean
- (3) If one of the six QC individual results is outside a $2S_i$ limit, reject run if (“out of control”):
- a) Outlier—One individual result is beyond the characterization mean $\pm 4 S_i$ or
 - b) R 4S Rule—Two or more of the within-run ranges in the same run exceed $4 S_w$ (i.e. 95 percent range limit)

S_i = Standard deviation of individual results (the limits are not shown on the chart unless run results are actually single measurements).

S_m = Standard deviation of the run means (the limits are shown on the chart).

S_w = Within-run standard deviation (the limits are not shown on the chart).

No results for a given analyte are to be reported from an analytical run that has been declared “out of control” for that analyte as assessed by internal (bench) QC. The initial limits are established by analyzing pool material in 20 runs and then are re-evaluated quarterly. When necessary, limits are updated to include more runs.

While a study is in progress, QC results are stored in a LIMS database. For runs that are not imported into the database (i.e., R&D, troubleshooting, research-type runs), QC results are stored electronically in the analyte-specific folder on the DLS network.

C. Sample QC Criteria

Each individual sample result is checked against established sample QC criteria limits to assess data quality. The method uses the following sample QC criteria:

- (1) CV among the 4 replicates per patient sample
 - a) If the CV of the 4 replicates is $\leq 15\%$, report the result
 - b) If the CV of the 4 replicates is $> 15\%$, evaluate whether deleting 1 replicate that appears to be an outlier (i.e., 6, 8, 8, 9 – 6 appears to be an outlier) reduces the CV to $\leq 10\%$; if it does, use the mean of the remaining 3 replicates for reporting with comment code 114; if the CV of the remaining 3 replicates is $> 10\%$, keep all 4 replicates with code 26 and repeat the sample
 - c) If there is no clear outlier (i.e., 6, 8, 8, 10), keep all 4 replicates; don’t report the result and use code 26; repeat the sample
 - d) Do not report results if < 3 replicates; use code 26 and repeat the sample
- (2) Sample raw concentration
 - a) The calibration curve range used in the folate MA is 0.05-1 nmol/L (raw concentration). If the raw concentration of a sample is < 0.05 nmol/L (lowest calibrator), the GEN5 software marks the result with a symbol; the sample will be repeated with less dilution in next run. If the raw concentration of a sample is > 1 nmol/L, the GEN5 software doesn’t calculate result; the sample will be repeated with higher dilution in next run.

For details, see “**4000_SOP sample QC Criteria**”.

11. Remedial Action if Calibration or QC Systems Fail to Meet Acceptable Criteria

- (A) Contamination is the most common problem with the microbiologic assay. Make sure chloramphenicol (antibiotic) is added into assay medium to stop other organisms from growing. Make sure supplies (beakers, cylinders, tubes, Digiflex, balance, pipettes, bench, lab coats, gloves, etc.) are not contaminated with folate. Preventative actions to avoid contamination are very important. Change of gloves after preparation of working standard is recommended. Also, preparation of standard stock solutions and washing of folate contaminated glassware should take place in a different place than the preparation of the daily assay.
- (B) Check errors in medium and reagents preparation, instrument performance (Janus, incubator and readers)
- (C) Check calibration of pipettes and automated liquid handler
- (D) Check errors in sample and standard dilution and dilution factors entered in Gen5
- (E) Review technical skills of analysts
- (F) If the steps outlined above do not result in correction of the “out of control” values for QC materials, consult the supervisor for other appropriate corrective actions

12. Limitations of Method; Interfering Substances and Conditions

- (A) This folate microbiologic assay measures total folate in serum or whole blood. It does not provide values for different forms of folate.
- (B) If samples contain antibiotics, the growth of the microorganism might be inhibited. This is typically seen by a non-linear relationship for samples with increased dilution.
- (C) There is still debate whether some non-folate compounds might stimulate the growth of the microorganism.
- (D) The most common causes of imprecision are intermittently inaccurate micropipettors or pipetting errors. Stock standards and specimens should be mixed thoroughly by vortexing before pipetting.
- (E) Assay medium and sodium ascorbate should be made fresh daily.
- (F) Hemolyzed serum samples may give falsely elevated values and results should be evaluated with caution.
- (G) Multiple freeze/thaws cycles for extended hours at room temperature will cause some degradation of folates.
- (H) The entire sample preparation and calibration should be conducted in yellow sub-dued light. Exposure to strong sunlight should be avoided since it may cause folate degradation.

13. Reference Ranges (Normal Values)

Clinical reference ranges reported for serum folate are 11-36 nmol/L with the microbiologic assay and 7-36 with the chemiluminescence assay [6]. Clinical reference ranges reported for RBC folate are 317-1422 nmol/L with the CPB radioassay [6].

The newest post-fortification reference ranges for the U.S. population generated with the microbiologic assay for NHANES 2005-2010 are shown below [7]. Pfeiffer et al. also reported microbiologic assay-

equivalent reference ranges for pre-fortification (NHANES 1988-1994) and early post-fortification (NHANES 1999-2004) periods, as well as reference ranges by population subgroups for all three time periods [7].

Serum folate: 12.7-104 nmol/L (2.5th -97.5th percentile; n = 23,528; NHANES 2005-2010)

RBC folate: 505-2,490 nmol/L (2.5th -97.5th percentile; n = 23,528; NHANES 2005-2010)

14. Critical Call Results (“Panic Values”)

Any samples with serum folate levels <7 nmol/L (<3 ng/mL), RBC folate levels <305 nmol/L RBC (<140 ng/mL) [8], are considered to require follow-up because of potential folate deficiency. Since survey data are transmitted approximately weekly to Westat, abnormal reports are automatically forwarded to the NCHS survey physician for follow-up. For smaller, non-NHANES studies, abnormal values are identified to the study principal investigator. Emails sent concerning abnormal results are maintained by the supervisor for the duration of the study. Most of these studies are epidemiological in nature.

15. Specimen Storage and Handling during Testing

Specimens should be allowed to warm to and be maintained at room temperature during preparation and testing and then returned to frozen storage (typically at ≤ -70 °C) as soon as possible

16. Alternate Methods for Performing Test of Storing Specimens if Test System Fails

While the LC/MS/MS method also determines total folate by calculating the sum of the individually measured forms, we generally do not use this expensive test as an alternate method for the microbiologic assay.

17. Test Result Reporting System; Protocol for Reporting Critical Calls (If Applicable)

Test results that are not abnormal are reported to the collaborating agency at a frequency and by a method determined by the study coordinator. Generally, data from this analysis are compiled with results from other analyses and sent to the responsible person at the collaborating agency as Excel file, generally through electronic mail or via ftp site.

For NHANES 1999+, all data are reported electronically weekly to Westat who then transfer the results to NCHS. For some smaller studies, hard copies of a data report are sent, as well as the results in electronic format.

18. Transfer or Referral of Specimens; Procedures for Specimen Accountability and Tracking

The LIMS database is used to keep records and track specimens for NHANES 1999+. If analyses are performed for smaller, non-NHANES studies, records may be kept in Excel files on the DLS network.

We recommend that records, including related QA/QC data, be maintained for 10 years after completion of the NHANES study. Only numerical identifiers should be used (e.g., case ID numbers). All personal identifiers should be available only to the medical supervisor or project coordinator. Residual serum from these analyses for non-NHANES studies are retained for at least 1 year after results have been reported and may then be returned or discarded at the request of the principal investigator. Very little residual material will be available after NHANES analyses are completed; however, residual serum is retained for at least 2 years after results have been publicly released; at that point, samples with sufficient volume (>0.2 mL) are returned to NHANES and samples with insufficient volume may be autoclaved.

The exact procedure used to track specimens varies with each study and is specified in the study protocol or the interagency agreement for the study. Copies of these documents are kept by the supervisor. In general, when specimens are received, the specimen ID number is entered into database and the specimens stored

in a freezer at -70 °C. The specimen ID is read off of the vial by a barcode reader attached to the computer used to prepare the electronic specimen table for the analytical system. When the analyses are completed, the DIF file containing the electronic copy of the results is loaded into the database, and the analytical results are linked to the database by ID number. The analyst is responsible for keeping records of specimens prepared incorrectly, those with labeling problems, and those with abnormal results, together with information about these discrepancies. In general, these are documented using codes in the LIMS.

19. Method Performance Documentation


Method performance documentation for this method including accuracy, precision, sensitivity, specificity and stability is provided in **Appendix A** of this method documentation. **The signatures of the branch chief and director of the Division of Laboratory Sciences on the first page of this procedure denote that the method performance is fit for the intended use of the method.**

20. Summary Statistics and QC Graphs

Please see follow page.

**2019-2020 Summary Statistics and QC Chart
LBXRBF (RBC folate (ng/mL))**

Lot	N	Start Date	End Date	Mean	Standard Deviation	Coefficient of Variation
3911	76	30JAN19	03APR20	589.7	35.2	6.0
3912	76	30JAN19	03APR20	990.5	42.4	4.3
3913	76	30JAN19	03APR20	1659.1	71.2	4.3

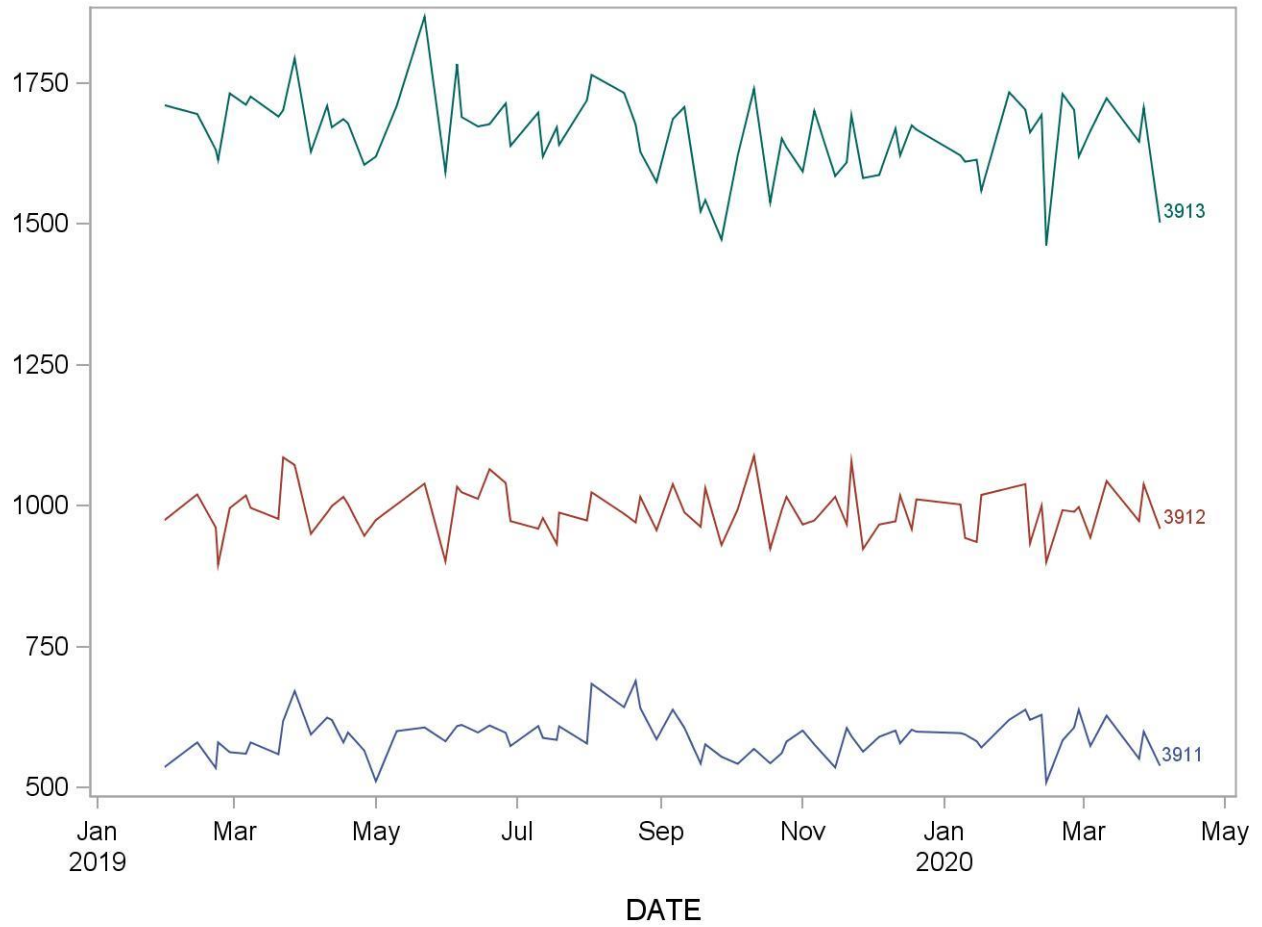

References

- (1) O’Broin S, Kelleher B. Microbiological assay on microtitre plates of folate in serum and red cells. *J Clin Pathol* 1992;45:344-7.
- (2) AM Molloy, JM Scott. Microbiological assay for serum, plasma, and red cell folate using cryopreserved, microtiter plate method. *Methods Enzymol* 1997;281:43-53.
- (3) Fazili Z, Pfeiffer CM, Zhang M, Jain R. Erythrocyte folate extraction and quantitative determination by LC/MS/MS: Comparison of results with microbiologic assay. *Clin Chem* 2005;51(12):2318-25.
- (4) Mastropaolo W, Wilson MA. Effect of light on serum B12 and folate stability. *Clin Chem* 1993;39(5):913.
- (5) Caudill SP, Schleicher RL, Pirkle JL. 2008. Multi-rule quality control for the age-related eye disease study. *Stat Med* 27:4094-4106.
- (6) Tietz NW. *Clinical Guide to Laboratory Tests*. Third Edition. 1995. WB Saunders Company.
- (7) Pfeiffer CM, Hughes JP, Lacher DA, Bailey RL, Berry RJ, Zhang M, Yetley EA, Rader JI, Sempos CT, Johnson CL. Estimation of trends in serum and RBC folate in the U.S. population from pre- to postfortification using assay-adjusted data from the NHANES 1988-2010. *J Nutr.* 2012;142:886-893.
- (8) Institute of Medicine, Food and Nutrition Board. *Dietary reference intakes: Thiamin, riboflavin, niacin, vitamin B6, folate, vitamin B12, pantothenic acid, biotin, and choline*. Washington, D.C.: National Academy Press; 1998.
- (9) Gupta VS, Huennekens M. *Arch. Biochem. Biophys.* 1967;120:712.
- (10) Pfeiffer CM, Zhang M, Lacher DA, Molloy AM, Tamura T, Yetley EA, Picciano M-F, Johnson CL. Comparison of serum and red blood cell folate microbiologic assays for national population surveys. *J Nutr.* 2011;141:1402-9.
- (11) Fazili Z, Pfeiffer CM, Zhang M. Comparison of serum folate species analyzed by LC-MS/MS with total folate measured by microbiologic assay and Bio-Rad radioassay. *Clin Chem* 2007;53:781-4.
- (12) Fazili Z, Whitehead Jr. RD, Paladugula N, Pfeiffer CM. A high-throughput LC-MS/MS method suitable for population biomonitoring measures five serum folate vitamers and one oxidation product. *Anal Bioanal Chem.* 2013;405:4549–60.
- (13) Fazili Z, Pfeiffer CM, Zhang M, Ram JB, Koontz D. Influence of 5, 10-methylene-tetrahydrofolate reductase polymorphism on whole blood folate measured by LC-MS/MS, microbiologic assay and BioRad assay. *Clin Chem* 2008;54:197-201.

Acknowledgements

We gratefully acknowledge the contributions of Mindy Zhang, MD for developing and validating this method and preparing the manuscript for this chapter and Christine M. Pfeiffer, Ph.D. for the assistance and advice provided.

Appendix A: Method Performance Documentation

Accuracy using Spike Recovery												
Recovery = (final concentration – initial concentration)/added concentration												
Recovery should be 85-115% except at 3*LOD where can be 80-120%												
Method name: Serum and WB folate by microbiologic assay												
Method #: 4000B and 4000S												
Matrix: Serum or WB												
Units: nmol/L												
Analyte: SFOL and WBF												
Replicate	Serum sample 1: LS14620					Serum sample 2: HS14622					Mean recovery (%)	SD (%)
	5MeTHF Spike concentration	Measured concentration		Mean	Recovery (%)	Spike concentration	Measured concentration		Mean	Recovery (%)		
	Day 1 (9/15/2017)	Day 2 (10/5/2017)				Day 1 (9/15/2017)	Day 2 (10/5/2017)				Day 1 (9/15/2017)	Day 2 (10/5/2017)
Sample	1	18.9	18.9			0	45.3	43.8				
	2	18.6	18.3	18.4			46.7	41.8	44.0		94.1	10.5
	3	17.6	18.0				45.7	40.8				
Sample + Spike 1	1	22.2	23.0			5	51.8	47.5				
	2	23.5	24.5	23.1	94.1		48.7	47.8	48.8	95.9		
	3	21.2	24.2				49.7	47.3				
Sample + Spike 2	1	29.0	27.2			10	56.6	54.8				
	2	28.0	27.6	27.5	90.5		56.2	52.9	55.4	113.6		
	3	26.7	26.3				57.7	54.0				
Sample + Spike 3	1	36.8	33.471			20	61.1	61.7				
	2	37.3	34.181	35	84.2		66.1	59.3	61	86.4		
	3	36.6	33.033				62.1	57.5				
Replicate	WB Sample 1: LB14810_MA					WB Sample 2: HB14812_MA					Mean recovery (%)	SD (%)
	5MeTHF Spike concentration	Measured concentration		Mean	Recovery (%)	Spike concentration	Measured concentration		Mean	Recovery (%)		
	Day 1 (1/18/2018)	Day 2 (1/25/2018)				Day 1 (8/23/2017)	Day 2				Day 1 (8/23/2017)	Day 2
Sample	1	217	201				707	602				
	2	207	202	208.1			673	636	646.8		97%	4%
	3	217	204				636	628				
Sample + Spike 1	1	454	429			22.1	930	862				
	2	495	398	446.9	98%		909	851	885.5	98%		
	3	487	419				886	875				
Sample + Spike 2	1	623	553			35.4	1012	988				
	2	661	591	604.4	102%		1093	1035	1033.7	99%		
	3	660	538				1081	993				
Sample + Spike 3	1	773	786			57.5	1303	1236				
	2	804	753	784	91%		1273	1140	1250	95%		
	3	793	794				1320	1226				

Accuracy Compared to Reference Material												
Mean concentration should be within $\pm 15\%$ of the nominal value except at $3 \times \text{LOD}$, where it should be within $\pm 20\%$												
Method name:	Serum and WB folate by microbiologic assay											
Method #:	4000B and 4000S											
Matrix:	Serum or WB											
Units:	nmol/L											
Analyte:	SFOL and WBF											
Date :	Feb-July 2017											
Reference material	Replicate	Nominal value	Measured serum folate concentration (nmol/L)									Difference from nominal value (%)
			Day 1	Day 2	Day 3	Day 4	Day 5	Mean	SD	CV (%)		
NIST 1955 Level 1	1	5.4	4.2	5.1	6.0	4.1	5.0	4.92	0.62	12.50	-8.8	
	2		4.3	5.5	5.4	4.5	5.1					
NIST 1955 Level 2	1	13.5	10.2	11.9	14.2	9.8	11.5	11.5	1.43	12.35	-14.5	
	2		10.5	12.1	13.5	10.8	11.1					
NIST 1955 Level 3	1	42.3	39.0	41.4	41.7	33.8	37.2	38.5	2.66	6.91	-9.0	
	2		36.0	41.9	39.3	37.3	37.3					
Reference material	Replicate	Nominal value	Measured serum folate concentration (nmol/L)									Difference from nominal value (%)
			Day 1	Day 2	Day 3	Day 4	Day 5	Mean	SD	CV (%)		
NIBSC 03/178	1	12.1	10.1	12.6	12.7	10.7	11.8	11.58	1.05	9.08	-4.3	
	2		10.7	11.9	13.2	10.6	11.6					
Reference material	Replicate	Nominal value	Measured WB folate concentration (nmol/L)									Difference from nominal value (%)
			Day 1	Day 2	Day 3	Day 4	Day 5	Mean	SD	CV (%)		
NIBSC 95/528	1	28.327	25.6	26.5	25.3	25.6	24.3	25.81	0.96	3.72	-8.9	
	2		25.6	25.1	27.8	26.6	25.7					

Precision						
Total relative standard deviation should be $\leq 15\%$ (CV $\leq 15\%$)						
Method name:	Serum and WB folate by microbiologic assay					
Method #:	4000B and 4000S					
Matrix:	Serum or WB					
Units:	nmol/L					
Analyte:	SFOL & WBF					
Serum quality material 1						
LS14620a_MA (nmol/L)						
Run	Result 1	Result 2	Mean	SS 1	SS 2	2*mean ²
1	18.6	19.8	19.25	0.36	0.36	740.74
2	20.7	20.0	20.34	0.14	0.14	827.51
3	18.8	19.7	19.23	0.19	0.19	739.78
4	22.8	18.8	20.77	3.97	3.97	862.83
5	19.4	20.1	19.76	0.11	0.11	781.23
6	19.6	18.6	19.13	0.25	0.25	731.88
7	20.8	21.3	21.04	0.08	0.08	885.62
8	20.6	21.6	21.10	0.22	0.22	890.59
9	20.0	21.5	20.76	0.60	0.60	861.54
10	21.7	19.5	20.58	1.23	1.23	846.66
Grand sum	404	Grand mean	20.2			
	Sum squares	Mean Sq Error	Std Dev	Rel Std Dev (%)		
Within Run	14.29	1.43	1.20	5.92		
Between Run	11.00	1.22	0.00	0.00		
Total	25.29		1.20	5.92		
Serum quality material 2						
HS14622a_MA (nmol/L)						
Run	Result 1	Result 2	Mean	SS 1	SS 2	2*mean ²
1	46.8	48.8	47.8	1.1	1.1	4570
2	49.4	50.9	50.1	0.5	0.5	5028
3	49.0	51.4	50.2	1.5	1.5	5038
4	53.5	50.0	51.8	3.0	3.0	5361
5	51.5	53.9	52.7	1.5	1.5	5560
6	52.2	49.2	50.7	2.1	2.1	5142
7	54.7	53.1	53.9	0.7	0.7	5811
8	50.0	50.4	50.2	0.0	0.0	5041
9	50.1	50.8	50.4	0.1	0.1	5087
10	51.4	51.0	51.2	0.0	0.0	5243
Grand sum	1018	Grand mean	50.9			
	Sum squares	Mean Sq Error	Std Dev	Rel Std Dev (%)		
Within Run	21.16	2.12	1.45	2.86		
Between Run	49.27	5.47	1.30	2.55		
Total	70.43		1.95	3.83		

WB quality material 1		LB14810a_MA (nmol/L)				
Run	Result 1	Result 2	Mean	SS 1	SS 2	2*mean^2
1	242	244	243.0	1.0	1.0	118098
2	235	218	226.5	72.3	72.3	102605
3	244	286	265.0	441.0	441.0	140450
4	257	269	263.0	36.0	36.0	138338
5	249	218	233.5	240.3	240.3	109045
6	239	273	256.0	289.0	289.0	131072
7	212	228	220.0	64.0	64.0	96800
8	275	264	269.5	30.3	30.3	145261
9	244	241	242.5	2.3	2.3	117613
10	251	241	246.0	25.0	25.0	121032
Grand sum	4930	Grand mean	246.5			
	Sum squares	Mean Sq Error	Std Dev	Rel Std Dev (%)		
Within Run	2402.00	240.20	15.50	6.29		
Between Run	5067.00	563.00	12.70	5.15		
Total	7469.00		20.04	8.13		

WB quality material 2		HB14812a_MA (nmol/L)				
Run	Result 1	Result 2	Mean	SS 1	SS 2	2*mean^2
1	744	717	730.5	182.3	182.3	1067261
2	696	680	688.0	64.0	64.0	946688
3	758	813	785.5	756.3	756.3	1234021
4	747	739	743.0	16.0	16.0	1104098
5	701	649	675.0	676.0	676.0	911250
6	740	774	757.0	289.0	289.0	1146098
7	626	630	628.0	4.0	4.0	788768
8	790	761	775.5	210.3	210.3	1202801
9	723	715	719.0	16.0	16.0	1033922
10	711	784	747.5	1332.3	1332.3	1117513
Grand sum	14498	Grand mean	724.9			
	Sum squares	Mean Sq Error	Std Dev	Rel Std Dev (%)		
Within Run	7092.00	709.20	26.63	3.67		
Between Run	42817.80	4757.53	44.99	6.21		
Total	49909.80		52.28	7.21		

Stability									
The initial measurement can be from the same day for all stability experiments.									
Freeze and thaw stability = Assess for a minimum of 3 freeze-thaw cycles; conditions should mimic intended sample handling conditions Describe condition: QC material thawed three times (3 hrs at room temperature) and re-frozen at -70°C (3 freeze-thaw cycles)									
Bench-top stability = Assess short-term stability for length of time needed to handle study samples (typically at room temperature) Describe condition: Diluted QC material in glass tubes at room temperature for 5 hrs, then pipetted into plates and incubated (9/26/17)									
Processed sample stability = Assess short-term stability of processed samples, including resident time in autosampler Describe condition: Plates were read after incubation according to protocol (initial measurement), then were kept at room temperature for ~7 hrs and read again for comparison. Reading date: serum (9/15/17); whole blood (9/11/17).									
Long-term stability = Assess long-term stability that equals or exceeds time between date of first sample collection and date of last sample analysis Describe condition: QC samples were stored at -70C since Aug 2014. Initial measurement: serum (4/21/15); whole blood (12/2/14). Stability was tested after 2-3 years: serum (9/1/2017); whole blood (8/11/17).									
All stability sample results should be within ±15% of nominal concentration									
Method name:	Serum and WB folate by microbiologic assay								
Method #:	4000B and 4000S								
Matrix:	Serum or WB								
Units:	nmol/L								
Analyte:	Total folate								
Serum Quality material 1					LS14620_MA				
	Initial measurement	Three freeze-thaw cycles	Initial measurement	Bench-top stability	Initial measurement	Processed sample stability	Initial measurement	Long-term stability	
Replicate 1	19.2	18.6	18.20	19.54	18.00	18.94	21.0	21.0	
Replicate 2	19.3	19.3	18.03	20.47	19.41	20.57	22.5	19.9	
Replicate 3	18.8	19.3	17.61	19.26	20.86	20.05	22.3	19.5	
Mean	19.1	19.1	17.95	19.8	19.42	19.9	21.94	20.11	
% difference from initial measurement	--	-0.1	--	10.1	--	2.2	--	-8.3%	
Serum Quality material 2					HS14622_MA				
	Initial measurement	Three freeze-thaw cycles	Initial measurement	Bench-top stability	Initial measurement	Processed sample stability	Initial measurement	Long-term stability	
Replicate 1	50.6	48.5	47.0	50.7	50.6	49.6	54.4	50.6	
Replicate 2	51.1	46.5	47.0	52.1	51.1	51.2	47.6	53.2	
Replicate 3	48.6	46.3	51.4	52.4	48.6	51.1	50.7	48.4	
Mean	50.1	47.1	48.5	51.7	50.1	50.6	50.9	50.7	
% difference from initial measurement	--	-5.9	--	6.7	--	1.0	--	-0.3%	
WB Quality material 1					LB14810_MA				
	Initial measurement	Three freeze-thaw cycles	Initial measurement	Bench-top stability	Initial measurement	Processed sample stability	Initial measurement	Long-term stability	
Replicate 1	242	283	283	229	274	262	221	274	
Replicate 2	297	261	265	251	264	273	256	264	
Replicate 3	251	233	259	240	248	262	258	248	
Mean	263	259	269	240	262	266	245	262	
% difference from initial measurement	--	-1.6%	--	-10.7%	--	1.4%	--	6.9%	
WB Quality material 2					HB14812_MA				
	Initial measurement	Three freeze-thaw cycles	Initial measurement	Bench-top stability	Initial measurement	Processed sample stability	Initial measurement	Long-term stability	
Replicate 1	734	768	825	667	799	764	710	799	
Replicate 2	824	688	754	756	759	745	723	759	
Replicate 3	791	747	753	710	728	652	711	728	
Mean	783	735	777	711	762	720	714.8	762.0	
% difference from initial measurement	--	-6.2%	--	-8.5%	--	-5.5%	--	6.6%	

LOD, specificity and fit for intended use			
Method name:	Serum and WB folate by microbiologic assay		
Method #:	4000B and 4000S		
Matrix:	Serum or WB		
Units:	nmol/L		
Analytes	Limit of Detection (LOD)	Interferences successfully checked in at least 50 human samples	Accuracy, precision, LOD, specificity and stability meet performance specifications for intended use
serum folate	2*	N/A**	Yes
WB folate	44*	N/A**	Yes
<p>* Raw folate concentration from diluted samples must be = or > lowest calibrator (0.05 nmol/L).</p> <p>The limit of detection of this method depends on the dilution factor used for samples. Samples with low folate concentrations are diluted less than samples with high folate concentrations. LOD values reported in the table are for the smallest possible dilution factor (1:20 for serum samples and 1:40 for whole blood hemolysate samples).</p> <p>** Growth inhibition can occur with this assay if the sample contains an antifolate (eg, methotraxate) or some antibiotics. The growth inhibition pattern can be recognized by the large difference in results from different dilutions. A repeat analysis is conducted to confirm the observed pattern and the result is set no reportable.</p>			

Appendix B: Information on Absorption Maxima, Absorption Coefficient, and Formulas to Calculate the Folate Concentration

Compound	Diluent for primary stock solution I	Absorption maximum (nm)	Molar extinction coefficient (L/mol*cm)	Reference	Molecular weight (g/mol)
5-MethylTHF	20 mM phosphate buffer with 0.1% cysteine (pH 7.2)	290	31,700	[9]	459.46

Additional information can be found in:

Blakley R.L. The biochemistry of folic acid and related pteridines. In: Neuberger A, Tatum EL, eds. Frontiers of biology. Amsterdam: North Holland Publishing Company, 1969:91–5.

Conversion factors from conventional (ng/mL) to SI units (nmol/L) for 5-MethylTHF is 2.176

Formulas to calculate the concentration of folate stock solutions based on molar absorptivity:

$$\text{Conc. (ppm or } \mu\text{g / mL)} = [\text{Absorbance (cm}^{-1}) \times \text{dilution} \times 1000 \times \text{MW (g/mol}^{-1})] / \epsilon_{\text{max}} (\text{Lmol}^{-1} \text{cm}^{-1})$$

$$\text{Conc. (}\mu\text{mol / L)} = [\text{Absorbance (cm}^{-1}) \times \text{dilution} \times 1000 \times 1000] / \epsilon_{\text{max}} (\text{Lmol}^{-1} \text{cm}^{-1})$$

ϵ_{max} = molar extinction coefficient and conc. = concentration

Example – 5-Methyltetrahydrofolic acid (5-MeTHF)

Abs	0.661
Dilution	10
ϵ_{max}	31,700
MW	459.4

$$\text{Concentration (ppm or } \mu\text{g/mL)} = 0.661 \times 10 \times 1,000 \times 459.4 / 31700 = \mathbf{95.79}$$


$$\text{Concentration (}\mu\text{mol/L)} = 0.661 \times 10 \times 1,000 \times 1,000 / 31700 = \mathbf{208.52}$$

Appendix C: Information on Response of Microorganisms to Different Calibrators and on Comparison of Microbiologic Assay to other Assays

Response of *L. casei* to different folate calibrators

As part of our assay validation, we investigated the response of the *L. casei* microorganism to different calibrators. We observed slight differences, with reduced forms (5MeTHF and 5FoTHF) giving a slightly higher response than folic acid (see Fig. 1). Since most of the folate in serum and whole blood is in the 5MeTHF form, we calibrate with 5MeTHF, not with folic acid, which is typically used by other labs, although some are using 5FoTHF as a calibrator. Calibration with 5MeTHF results in ~25% lower values than calibration with folic acid. This effect has been documented in a paper that describes data from 3 different microbiologic assays applied to a subset of NHANES 2007-2008 [10].

Fig. 1: Response of *L. casei* to different folate calibrators

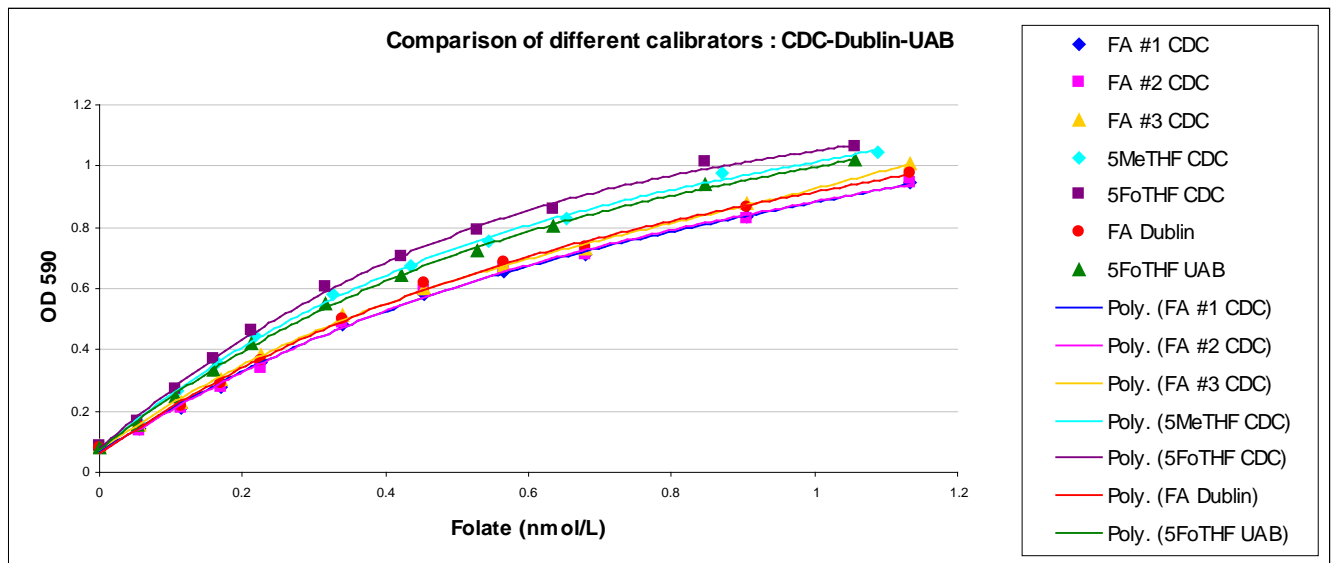

Comparison of microbiologic assay to other assays

We have performed in-house comparisons of the microbiologic assay, the LC/MS/MS assay, and the BioRad assay. Microbiologic assay is still considered an accurate “reference point” for total folate. For serum samples, there is excellent correlation between the three assays. The microbiologic assay produces results that are within $\pm 10\%$ of the LC-MS/MS results [11, 12]. For whole blood samples, there is excellent correlation and agreement between the microbiologic assay and the LC/MS/MS as long as folate polyglutamates have been properly deconjugated to monoglutamates, however overall the microbiologic assay gives results that are 10-25% higher than LC-MS/MS results [13]. We have shown that calibration of the microbiologic assay with 5-methylTHF produces results that are ~25% lower than calibration with folic acid [10].

Appendix D: Ruggedness Testing

This method has undergone a series of in-house ruggedness testing experiments designed to assess how much method accuracy changes when certain experimental parameters are varied. A total of five parameters judged to most likely affect the accuracy of the method have been identified and tested. Testing generally consisted of performing replicate measurements on a test specimen with the selected parameter set at a value substantially lower and higher than that specified in this method while holding all other experimental variables constant. The ruggedness testing findings for this method are presented below. Please refer to the DLS Policies and Procedures Manual for further information on ruggedness testing.

A. Turbidity reading of mixed microplates

(1) Principle: The turbidity reading of mixed microplates has to occur as long as the microorganism is in suspension and the sample is homogenous. Standard protocol: Incubate plates at 37 °C for ~45 h, mixed them with a shaker for ~15 min, then by hand inversion ~20 times; open the sealing membrane, get rid of bubbles, and then read the plates at ~1 min, as long as the microorganism is still in suspension.

(2) Purpose: To vary the time window within which mixed plates are read.

(3) Findings:

	Lower level	Method specifies	Higher level a	Higher level b	Higher level c
Reading time (min)	0.5	1	2	3	4
Mean serum folate (nmol/L) ¹	20.3	18.6	20.6	20.4	19.8
Difference from 1 min for serum	9%		11%	10%	6%
Mean WB folate (nmol/L) ¹		354	370	364	364
Difference from 1 min for WB			5%	3%	3%

¹Average folate concentration 3 QC levels, each in duplicate

(4) Conclusion: Varying the reading time between 0.5 min and 4 min after plate mixing does not appear to affect serum or WB folate concentrations.

B. Delayed turbidity reading of microplates

(1) Principle: Microplates are usually read directly after the ~45 h incubation. In special situations, the plate reading has to be delayed and plates are stored at RT or 4 °C prior to reading.

(2) Proposal: To assess effects of delayed reading plates. Plates were first read as specified in the method and then re-read after some delay.

(3) Findings:

	Method specifies	High level a	High level b
Reading time point	Directly after ~45 h incubation	After storing at 4 °C for 4 days	After storing at room temperature overnight
Mean serum folate (nmol/L), N = 44	37.0	35.2	
Difference		-5%	
Mean serum folate (nmol/L), N = 92	38.6	39.4	
Difference		2%	
Mean serum folate (nmol/L), N = 4	12.6		12.2
Difference			-3%


(4) Conclusion: Delaying the plate reading does not appear to affect serum folate concentrations, as long as the plate is stored overnight at room temperature or up to 4 days at 4 °C.

C. Sodium ascorbate concentration

(1) Principle: Sodium ascorbate (SA) is an antioxidant. It is used at 0.5% to dilute samples and calibrators and to make up the final volume in microplates.

(2) Proposal: To assess the potential protective effects of higher SA concentration.

(3) Findings:

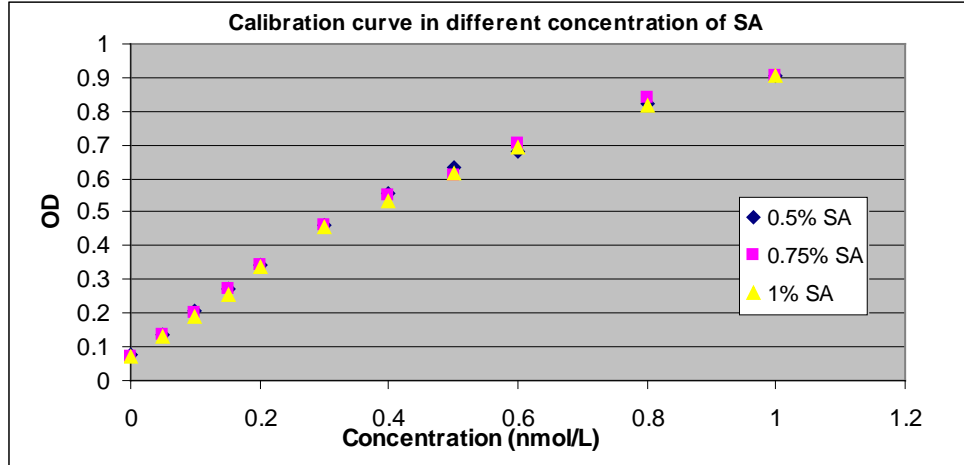


	Method specifies	Higher level a	Higher level b
SA concentration	0.5%	0.75%	1%
Mean serum folate (nmol/L), N = 13	40.3	40.1	43.1
Difference		0%	9%
Mean WB folate (nmol/L), N = 16	448	428	427
Difference		-4%	-5%

(4) Conclusion: Increasing the sodium ascorbate concentration from 0.5% to up to 1% does not appear to provide a protective effect to serum and WB folate concentrations, or to the 5-methyltetrahydrofolate standard in the calibration curve.

D. Stability of diluted calibrator

- (1) Principle: The 5-methyltetrahydrofolate calibrator has to be diluted before it can be pipetted into microplates. This compound is known to be sensitive to decomposition. The pipetting is therefore usually started within 30 min of calibrator preparation.
- (2) Proposal: To vary the length of time at which the diluted calibrator is kept before starting to pipette into microplates.
- (3) Findings: As long as pipetting is started within 30 min of preparing the diluted calibrator, calibration curves are very reproducible (see graph below). However, when the pipetting is delayed to 2 h and 4 h, the calibration curves show quite a bit of variation and are hard to interpret.

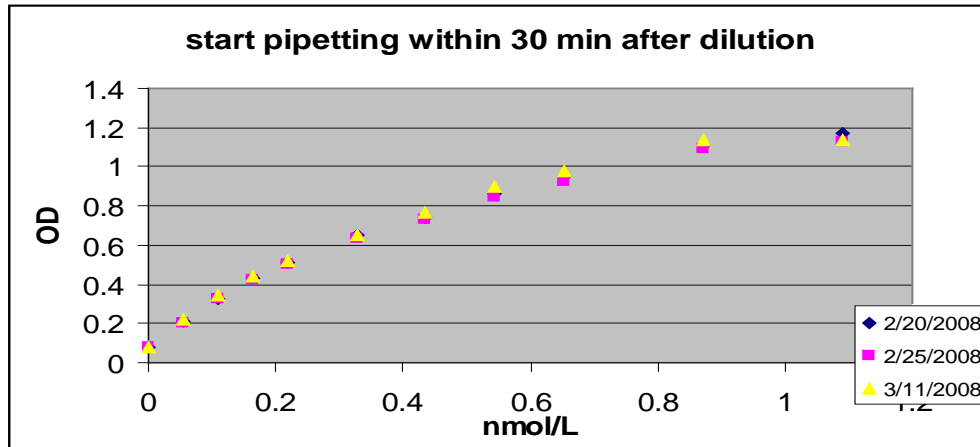

- (4) Conclusion: The pipetting of the diluted calibrator should be started within 30 min of preparation to generate stable and reproducible calibration curves.

E. Incubation time for microorganism growth

- (1) Principle: The microorganism requires ~42-45 h of incubation at 37 °C for reproducible growth.
- (2) Proposal: To vary the incubation time and evaluate whether shorter incubation (only 1 day) or longer incubation (a few extra hours beyond the regular time) is possible.

(3) Findings:


Serum	Shorter Incubation Time	Mean Diff (SD)	Longer Incubation Time	Mean Diff (SD)
4/28/2008 N=6	24h vs. 47h	3.0% (10.5%)		
4/29/2008 N=5	24h vs. 48h	-5.2% (11.5%)		
4/29/2008 N=10	24h vs. 48h	-8.1% (9.2%)		
9/8/2008 N=46	24h vs. 47h	7.4% (12.9%)	50h vs. 46h	-1.9% (7.3%)
9/23/2008 N=6	24h vs. 42h	4.2% (13.4%)	49h vs. 42h	-16.7% (7.9%)
11/19/2008 N=45	24h vs. 42h	3.3% (14.0%)	49h vs. 42h	-3.5% (7.4%)

WB	Shorter Incubation Time	Mean Diff (SD)	Longer Incubation Time	Mean Diff (SD)
4/28/2008 N=6	24h vs. 47h	1.6% (12.2%)		
8/27/2008 N=46			51h vs. 46h	-8.9% (7.6%)
9/2/2008 N=113			49 h vs. 47h	-6.5% (5.5%)
9/2/2008 N=17	24h vs. 41h	39.2% (16.6%)	48h vs. 41h	5.6% (6.9%)
9/3/2008 N=91			49h vs. 45h	-8.5% (7.4%)
1/14/2009 N=23	24h vs. 45h	-9.7% (13.4%)	48h vs. 45h	1.3% (5.7%)

- (4) Conclusion: Incubating plates for a few extra hours beyond the regular time appears to be generally ok, but should be avoided if possible. Plates should not be incubated for only 24 h due to inconsistent results and high variability from sample to sample (as reflected in the relatively large SD).

F. Stability of three frozen reagents stored at -70°C

- 1) Principle: The extended stability of frozen stock solutions of ascorbic acid, chloramphenicol, and manganese sulfate stored at -70°C needs to be determined.
- 2) Proposal: Compare stock solutions kept at -70°C for ~6 months vs. ~2 years
- 3) Findings: The comparison data (from 1 run with 38 WB samples) showed high correlation $r=0.9971$ and a low mean difference of 1.3% (SD 4.7%).

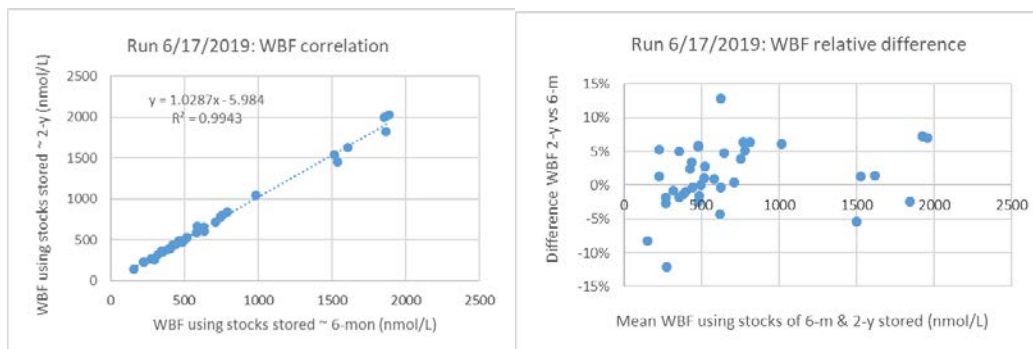

- 4) Conclusion: The three reagent stock solutions are stable up to 2 years when stored at -70°C. We changed the three reagent stocks storage date from 0.5 year to 2 years in June 2019.

G. Stability of calibrator stored at -70°C

- 1) Principle: The extended stability of 5MeTHF calibrator stock solution (1 umol/L) stored at -70°C needs to be determined.
- 2) Proposal: Perform 4 runs to compare 5MeTHF calibrator stored at -70°C for 1.5 years vs. a newly-prepared lot (~ 3 months storage at -70°C).
- 3) Findings:

Mean relative difference of WBF and SFOL results between Std Lot 2018-02 (~1.5 year storage) and Std Lot 2019-07 (3-m storage)					
	Date		N	Mean difference	SD
Run 1	9/17/19	WBF	23	0.9%	1.0%
Run 2	9/23/19	WBF	23	5.3%	0.1%
Run 3	9/30/19	WBF	55	6.9%	1.2%
Run 4	10/16/19	SFOL	25	-6.4%	1.9%
Mean difference				1.7%	1.1%

- 4) Conclusion: The comparison between 1.5 year and 3 months storage time showed acceptable difference (1.7%). We changed the calibrator storage date from 0.5 year to 1.5 years in Oct 2019.